

One Call

One Source

A World of
Drug Abuse
and Alcohol
Testing

MP RAPID TESTS

A WORLDWIDE LEADER IN ON-SITE DRUG TESTING

- *U.S. manufactured*
- *Simple one-step procedure*
- *Accurate results in 3 - 5 minutes*
- *Clear and sensitive color change*
- *Customizable panels up to 14 tests*
- *Comprehensive laboratory services available*
- *Extensive shelf life*
- *New tests coming soon!*

- FDA 510 K Cleared
- ISO Certified Manufacturer
- CE Marked

Ask about our
**SPECIAL
OFFERS**



www.mpbio.com

Contact us at 1.888.842.8378 for a Free Sample & Quote!



MP™ RAPID DIAGNOSTICS Rapid Drug Test

- Simple one-step procedure
- Precise test results within 5 minutes
- Up to 10 different drugs on one cup
- Multiple Formats
- U.S. Manufactured
- A quality branded product at a competitive price



Verifiable drugs:

| | |
|-------------------------------|--------|
| Amphetamine | AMP |
| Barbiturate | BAR |
| Benzodiazepine | BZD |
| Cocaine | COC |
| Methadone | MTD |
| Methamphetamine | M-AMP |
| Opiate I (low cutoff value) | OPI |
| Opiate II (high cutoff value) | OPI-II |
| Phencyclidine | PCP |
| THC | THC |
| Buprenorphine | BPNP |
| Oxycodone | OXY |
| Tricyclic antidepressants* | TCA |
| EDDP* | EDDP |
| MDMA* | MDMA |

*510K pending

The MP™ Rapid Drug Test quickly and reliably identifies various drugs and their major metabolites by their presence in urine.

The immunochemical test strip principle employed combines high sensitivity with great ease of use.

The MP™ Rapid Drug Test is available in a test cup or strip/card format, allows for the identification of a single drug or up to 14 different drugs.

As the direct manufacturer MP Biomedicals guarantees the highest quality and functionality of its products.

MP Biomedicals continues to invest in new product development ensuring its customers new assays and product improvements to meet their changing needs.

To order call 1.888.842.8378 or fax (949) 859.5010.



Quality That Counts

The test is easy:

MP™ rapid drug tests are stored and used at room temperature.

Step 1: First remove the protective cap from the test card.

Step 2: Hold the test strips in the urine sample for 10 to 20 seconds, then replace the protective cap.

Step 3: The results can be read at 5 minutes.

The advantage to you:

We can produce any combination of up to 10 tests individually for you at an attractive price.







Analyzing the test results:

NEGATIVE: There is a color line in both the control zone (C) and in the test zone (T). No drugs are present, or the concentration is below the cutoff limit.

NOTE: Line intensity may vary. The presence of any line, whether faint or broken, is to be interpreted as a negative result.

POSITIVE: A color line appears only in the control zone (C). There is no line in the test zone (T). The concentration is above the cutoff limit.

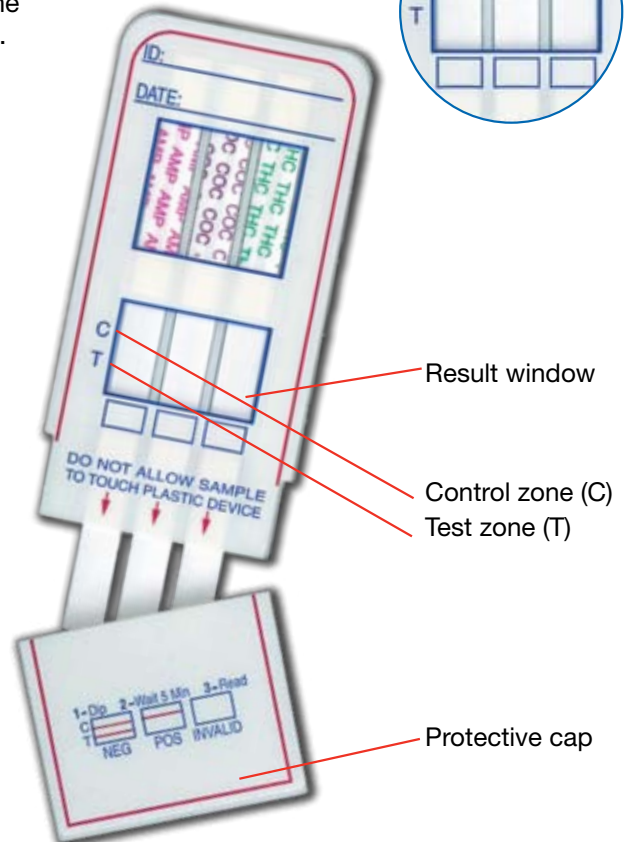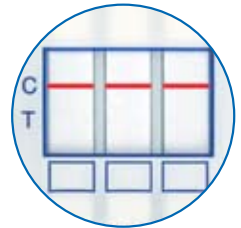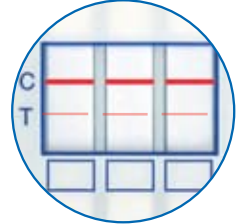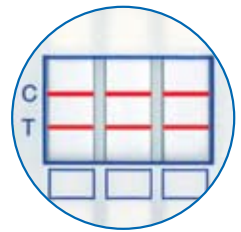

To order call 1.888.842.8378 or fax (949) 859.5010.

One Call

MP RAPID TESTS

One Source

A WORLDWIDE LEADER IN
ON-SITE DRUG TESTING

A World of
Drug Abuse
and Alcohol
Testing



www.mpbio.com

MP Biomedicals, 3 Hutton Centre Drive, Santa Ana, CA 92707



MP RAPID TESTS

- U.S. manufactured
- Simple one-step procedure
- Accurate results in 3 - 5 minutes
- Clear and sensitive color change
- Customizable panels up to 14 tests
- Comprehensive laboratory services available
- Extensive shelf life
- New tests coming soon!

Ask about our
**SPECIAL
OFFERS**

- FDA 510
K Cleared
- ISO Certified
Manufacturer
- CE Marked



CE

www.mpbio.com

Contact us at 1.888.842.8378 for a Free Sample & Quote!



To request product samples and enter to win a Starbucks Gift Card,
fax this completed sheet to 949.859.5010

or mail to: MP Biomedicals, Attn: Rapid Diagnostics, 3 Hutton Centre Drive, Santa Ana, CA 92707

MP™ Rapid Drug Test

Yes, please send me a free trial sample of the following items:

- | | | |
|--|--|---|
| <input type="checkbox"/> Urine DOA Tests | | |
| <input type="checkbox"/> Saliva Alcohol Test Strip | | |
| <input type="checkbox"/> Integrated DOA cup | | |
| <input type="checkbox"/> Amphetamine | <input type="checkbox"/> Cocaine | <input type="checkbox"/> Opiate (I) |
| <input type="checkbox"/> Barbiturate | <input type="checkbox"/> Marijuana | <input type="checkbox"/> Opiate (II) |
| <input type="checkbox"/> Benzodiazepine | <input type="checkbox"/> Methadone | <input type="checkbox"/> Oxycodone |
| <input type="checkbox"/> Buprenorphine | <input type="checkbox"/> Methamphetamine | <input type="checkbox"/> PCP |
| | | <input type="checkbox"/> Saliva Alcohol |



Please indicate the tests required - don't forget to include your name and address!

Answer and win!

In order to continue MP Biomedicals' endeavor to offer an excellent service, would you answer a few questions about drug screening so that we can respond to your needs in the future?

All respondents sending in a completed reply will be entered in a free prize drawing to win a \$100 Starbucks Gift Card! Good luck!



How many patients are you treating at present?

- None <10 10-50 >50

Do you carry out rapid drug tests on your patients to monitor the treatment?

- Yes No

How often do you monitor them?

On average every _____ weeks

How often do you discover use of other drugs?

In about _____ % of cases

How many drugs do you test for as a rule?

Approx. _____

- I am interested in other MP™ Rapid Tests (Adulteration Strips).

Contact: _____

Practice/Clinic: _____

Address: _____

Phone Number: _____

Stamp/signature:

To order call 1.888.842.8378 or fax (949) 859.5010.



