

HTLV-I/II ELISA 4.0

Trusted By Clinicians Since 1990

MPBROL-HTE202603-02

Detecting the Presence of Antibodies to Human T-Lymphotropic Virus Type I and II

Double Antigen Sandwich Assay


Utilizes Recombinant Antigens

High Sensitivity and Specificity


Short Turn-Around-Time

Potential Use for HTLV-III/STLV-III

Assay Procedure

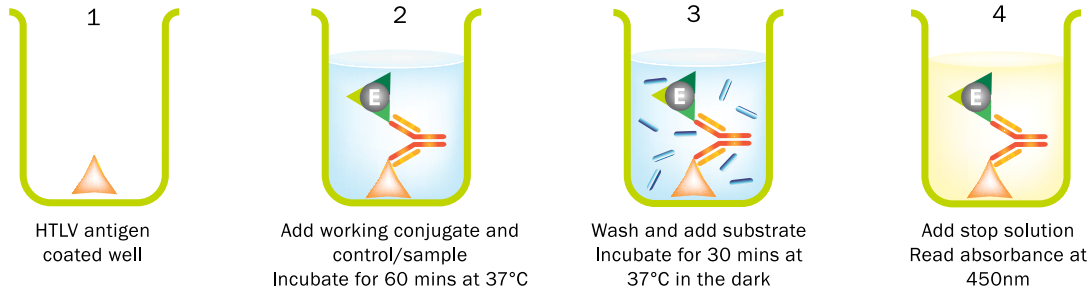

Legend

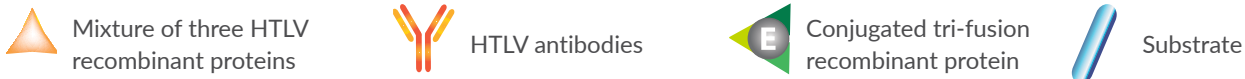

Sensitivity

Sample Type	No. of Sample	Reactive	Confirmed Positive	Sensitivity
HTLV-I	371	371	371	100%
HTLV-II	134	134	134	100%
HTLV I/II co-infection	6	6	6	100%
HTLV seropositive	4	4	4	100%

Specificity

Sample Type	No. of Sample	Non-Reactive	Specificity
Blood donor	4999	4990	99.82% (4990/4999)

Evaluation Performed Using Automated ELISA Analyzer With Fresh Samples From Unselected Donors

System	No. of Sample	Initial Reactive	Repeatedly Reactive	Confirm Positive	Specificity
BEP3	2024	4	3	0	99.85%
Summit	2024	6	5	0	99.75%

Order Information

Description	Pack Size	Cat. No.
HTLV-I/II ELISA 4.0	1 Plate	23081-096
	2 Plates	23081-192
	5 Plates	23081-480



MP BIOMEDICALS

AMERICAS: 800.854.0530 | custserv.na@mpbio.com

APAC: +65 6775.0008 | custserv.ap@mpbio.com

EUROPE: 00800.7777.9999 | custserv.eur@mpbio.com

Learn more at www.mpbio.com

